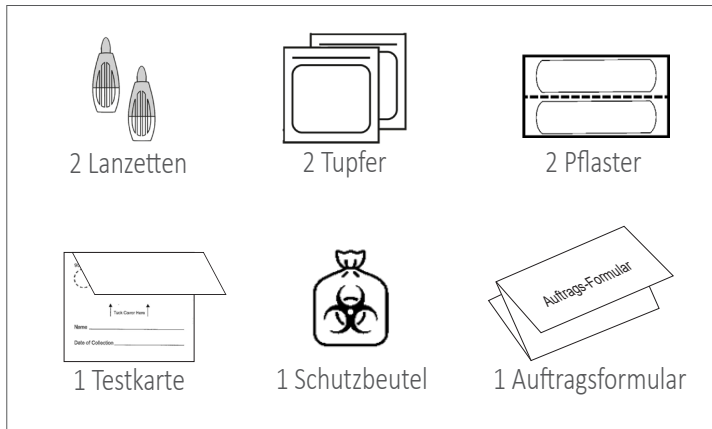
P88-DIY

Lebensmittel Antigen-Test

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1 INHALT

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P88-DIY

Diatary Antigen Test Sample Collection Instructions

1 KIT CONTENT

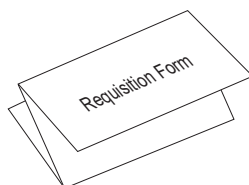
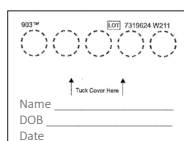
Please read the instructions carefully before you get started

2 Tips for good blood flow

- Drink enough liquid 2h before sample collection (400ml)
- Only warm hands guarantee good blood flow. Before taking the test increase your blood circulation by rubbing your hands, jumping, walking, or any other physical exercise you prefer.
- It might be necessary to massage your finger to increase blood flow. Avoid squeezing your finger too much. Exuding tissue fluid could dilute the blood sample and lead to false results.
- When collecting blood, hold pricked finger below height of heart.

3 Fill out Test Card and Requisition Form


Write your name, date of birth and date of sample collection on the test card. Fill out the requisition form completely.


incomplete labeled samples must not be processed and will be rejected

4 Preparation

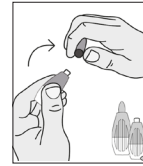
Twist off
Lancet cap


Have
Test Card ready

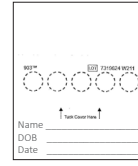Unpack
Gauze

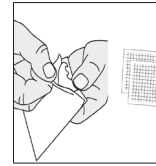Unpeel one side
of bandage

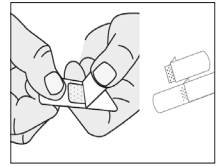5 Step-by-Step Instructions

Wash your Hands

thoroughly with warm water and soap. Rinse off the soap with plenty of water before drying them with a clean towel

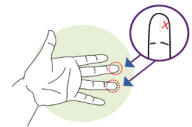
Warm your hands

Warming your hands is important for a good blood flow.

Select finger

Middle or ring finger are the best choice. Place lancet off-center of finger midline

Prick your finger

by laying your hand on a hard surface. Position the lancet on the chosen location. Apply firm pressure until click is heard.

Wipe away first blood drop

with the gauze. A blood drop may take up to 20sec. to form. If needed, gently massage pricked finger upward until drop forms.

Gently massage finger to produce a large droplet of blood

Then touch the droplet to one circle of the collection card
DO NOT touch the card with your finger.

Fill all five circles completely.

NOTE: You may find that wiping your finger with the gauze helps with blood flow.

Set Blood Spot Card down

on hard surface and apply bandage to your finger

Unacceptable collection

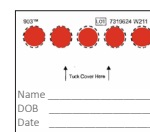
Circles are incorrectly filled


6 Shipping

Allow blood card to dry for 15min


at room temperature. DO NOT use artificial heat like hair dryer, heating, microwave

close and tuck over
Blood Spot Card

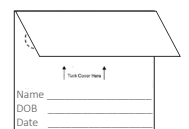place card into biohazard bag

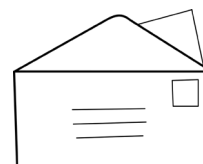put Requisition form and
biohazard bag with your
sample into envelope